

Simplified access in less than 30 seconds!

180°

Assistance in accessibility, promoting swimming for people with disabilities.

Optional footrest

unijet

- No engine
- With simple anchor
- Without power supply or battery

Manufacture
HEXAGONE

- Space saving
- Simplified access
- Apave Certification

unijet

Lift

www.myhexagone.com



+33 (0)1 34 34 11 55

Solution for disabled access



Minimal
maintenance

HYDRAULIC LIFT



Hydraulic disabled lift for deck level pools.

Works using the mains water pressure supplied from a hose to lift the hydraulic cylinder (minimum pressure: 3 bars).

- **Easy to use:** Control valve to raise and lower the lift, can be controlled by user or staff. No electrics or battery required.
- **Easy to handle:** Transportation wheels to easily move the Unijet from storage to the pool.
- **Easy to install:** Only one hole is needed to be drilled to install the Unijet, to secure it against tipping

Security :

- With fixation, tipping is not possible
- Optional footrest for additional security and comfort
- A safety pin locks the seat in the high position.



Control handle



Safety wheel in the ground



Optional
foot rest

TECHNICAL DATA

Hoisting load max.

Weight

Power supply

Structure

Seat

Warranty

Fixation

Seat course

Dimensions (WxDxH)

Depth of basin

Unijet

Up to 120 kg

55 kg

Hydraulic

316 L painted stainless steel

Rigid shell

2 years


Coring

1,20 m





70 x 70 x 170 cm

1 meter minimum

How to fix by coring in 5 steps:

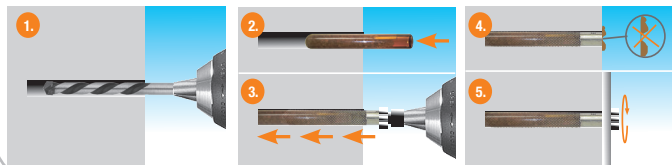

1. Drill and clean hole.
2. Insert the resin bulb.
3. Push the bit with the bit attached to the drill. (No hammer).
4. Clean excess resin.
5. Plate and tighten.
Leave to dry 1 hour before use.

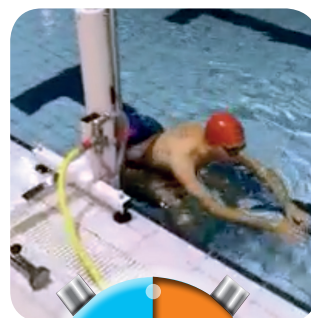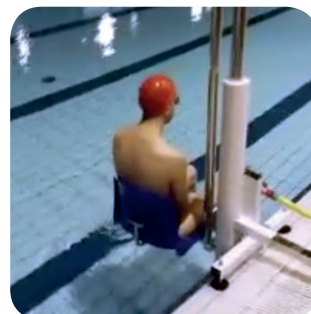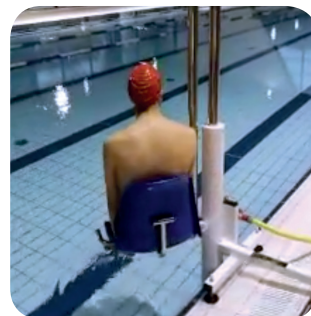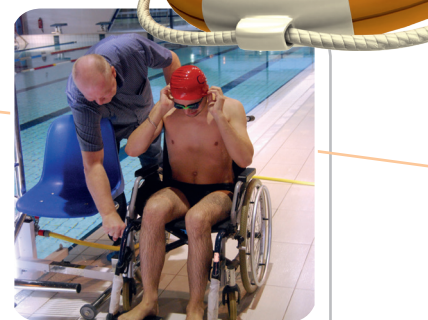

www.myhexagone.com



+33 (0)1 34 34 11 55

Solution for disabled access